




COVID-19 IgM/IgG Rapid Test

VivaDiag™ COVID-19 IgM/IgG Rapid Test is an *in vitro* diagnostic test for the qualitative determination of COVID-19's IgM and IgG antibodies in human whole blood (from vein or fingertip), serum or plasma.

Qualitative Fast Reliable



Steps for a single test

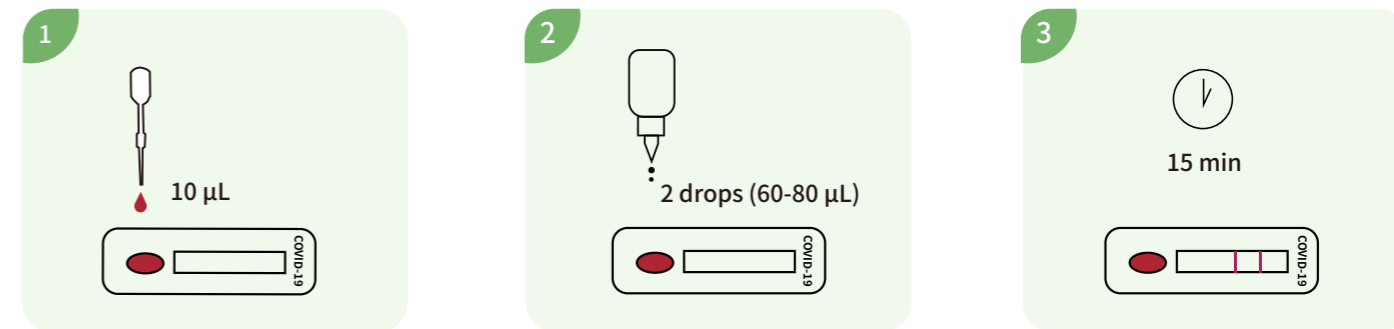


1. Add 10 µL specimen

2. Add 2 drops of buffer

3. Read the test result

Note: please make sure read the result at 15 minutes. If at other time, it may cause wrong results

Interpretation of results

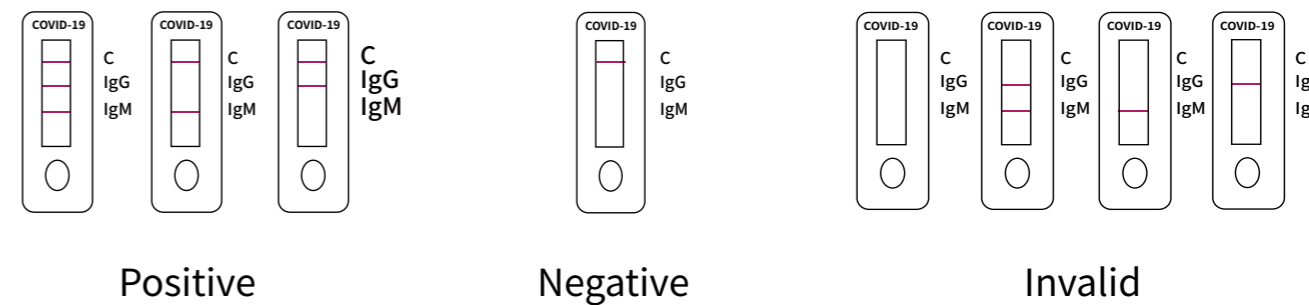

Positive

Negative

Invalid

Specification

Product Name	VivaDiag™ COVID-19 IgM/IgG Rapid Test
Test Principle	Colloidal gold
Sample Type	Whole blood (from vein or fingertip), serum or plasma
Sample Volume	10 µL
Test Time	15 min
Operation Temperature	18-25°C
Storage Temperature	2-30°C
Shelf Life (Unopened)	12 months